



# Auracast Listening Guide for **Beltone** Hearing Aids

---



Learn how to listen  
to Auracast broadcasts

Start

< Back

# Choose your device

For **Auracast** Hearing Aids:

- *Commence* • *Envision* • *Serene*



Use  
**Beltone app**



For **non-Auracast** Hearing Aids:



Use an  
**Auracast receiver**



Use  
**Auracast headphones**



< Back

Cover

# Using Beltone App

**1** Open **Beltone** app on your phone.




Next

< Back

Cover

② Tap **Find a broadcast**.

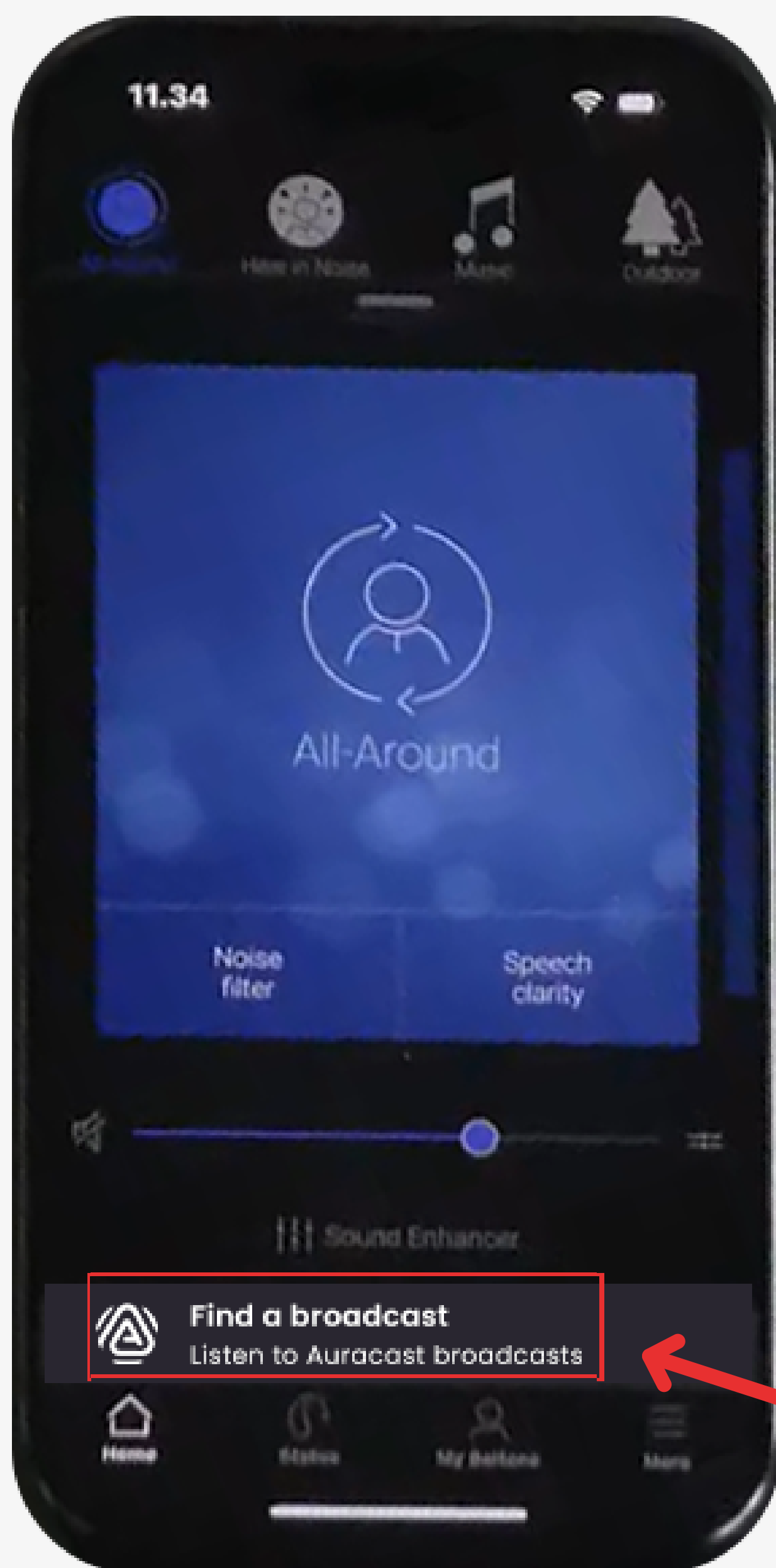


Next

< Back

Cover

③ Select a broadcast.

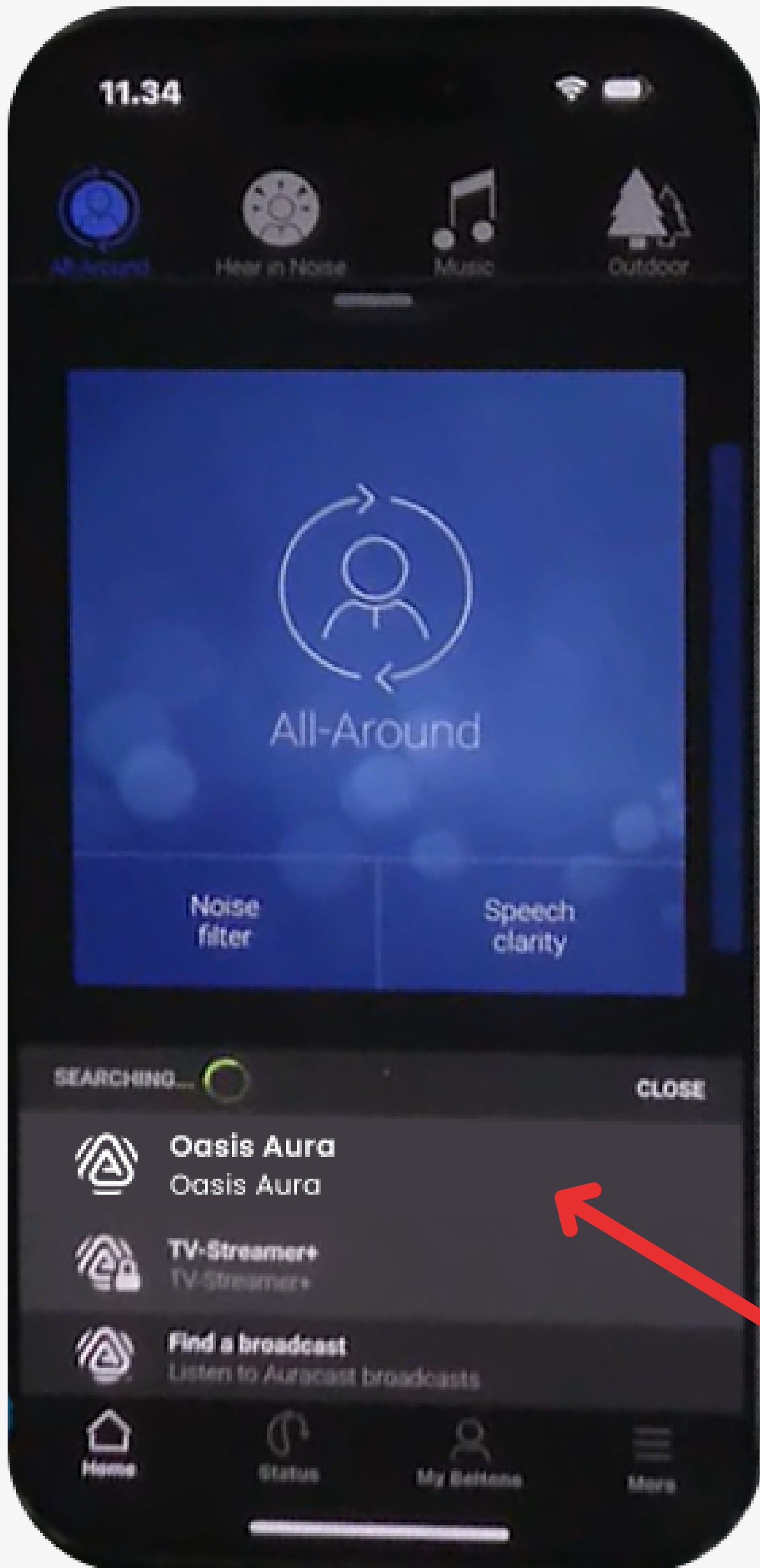


Next

< Back

Cover

4 Adjust volume.

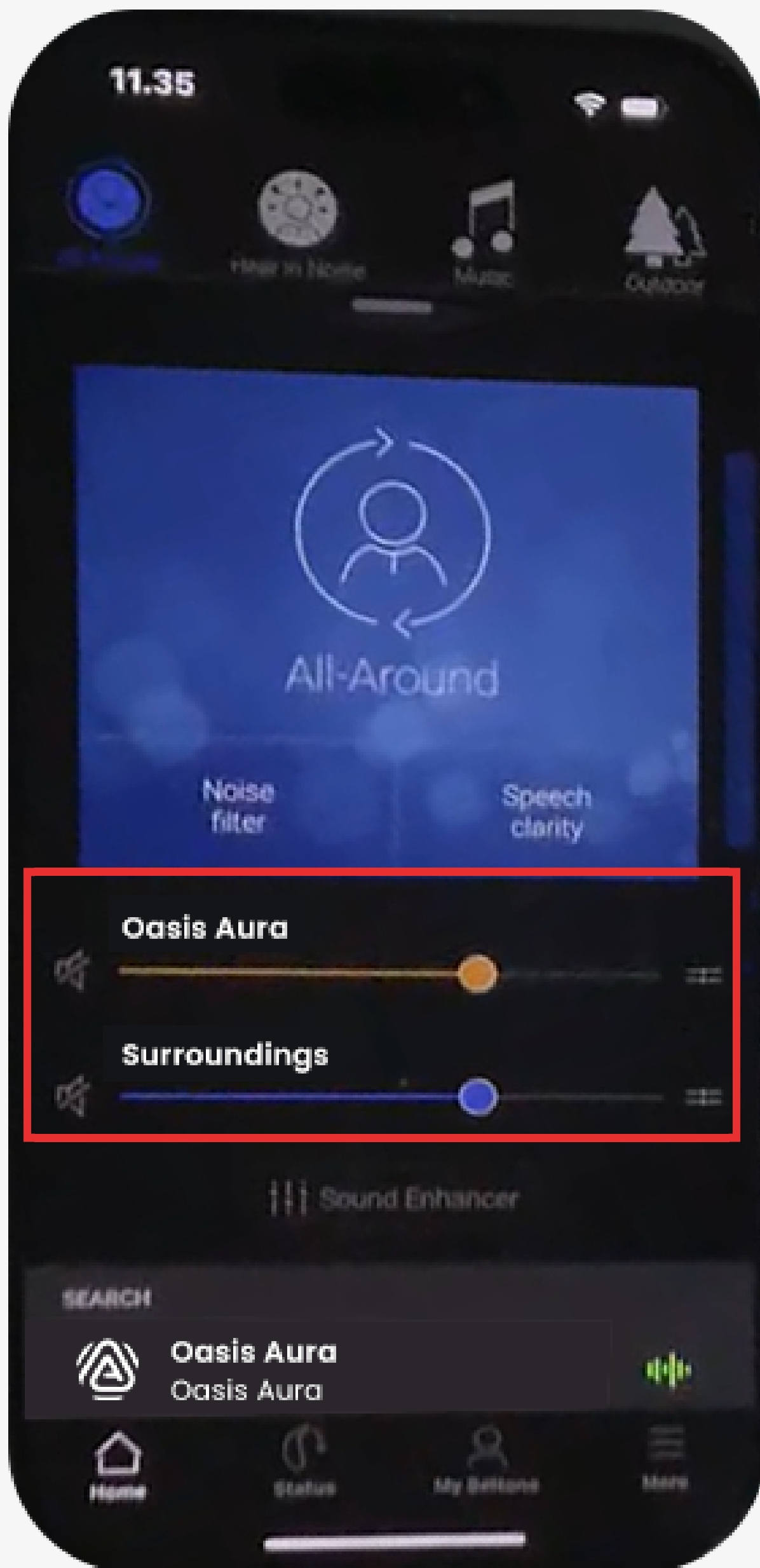

Next

< Back

Cover

## Using an Auracast Receiver

Recommended receiver: [AuraClip](#)



Next

< Back

Cover

## Using Auracast Headphones

You can wear Auracast headphones over your hearing aids, or use them without hearing aids.

Example: [Aria Aura](#)

Option a. Wear Over Your Hearing Aids



Option b. Use Without Hearing Aids



Next

< Back

Cover

# Troubleshooting

## Common Issues

---

- No sound
- Not hearing clearly
- Hearing echo or delay

 [More FAQ & Help](#) ↗



*Still need help?*

[Contact support](#) ↗

Next

< Back

Cover

# Rate This Guide

Your feedback helps us improve this guide for everyone.

It only takes a moment.

**Rate This Guide**