



# Auracast Listening Guide for **Bernafon** Hearing Aids

---



Learn how to listen  
to Auracast broadcasts

Start

< Back

# Choose your device

For **Auracast** Hearing Aids:

*Encanta*



Use  
**Bernafon app**



For **non-Auracast** Hearing Aids:



Use an  
**Auracast receiver**



Use  
**Auracast headphones**



< Back

Start  
Over



# Using Bernafon App

- 1 Open **Bernafon** app  
(*Version 1.6.1+*) on your phone.




Next

< Back

Start  
Over



② Tap ... (More Options) button.

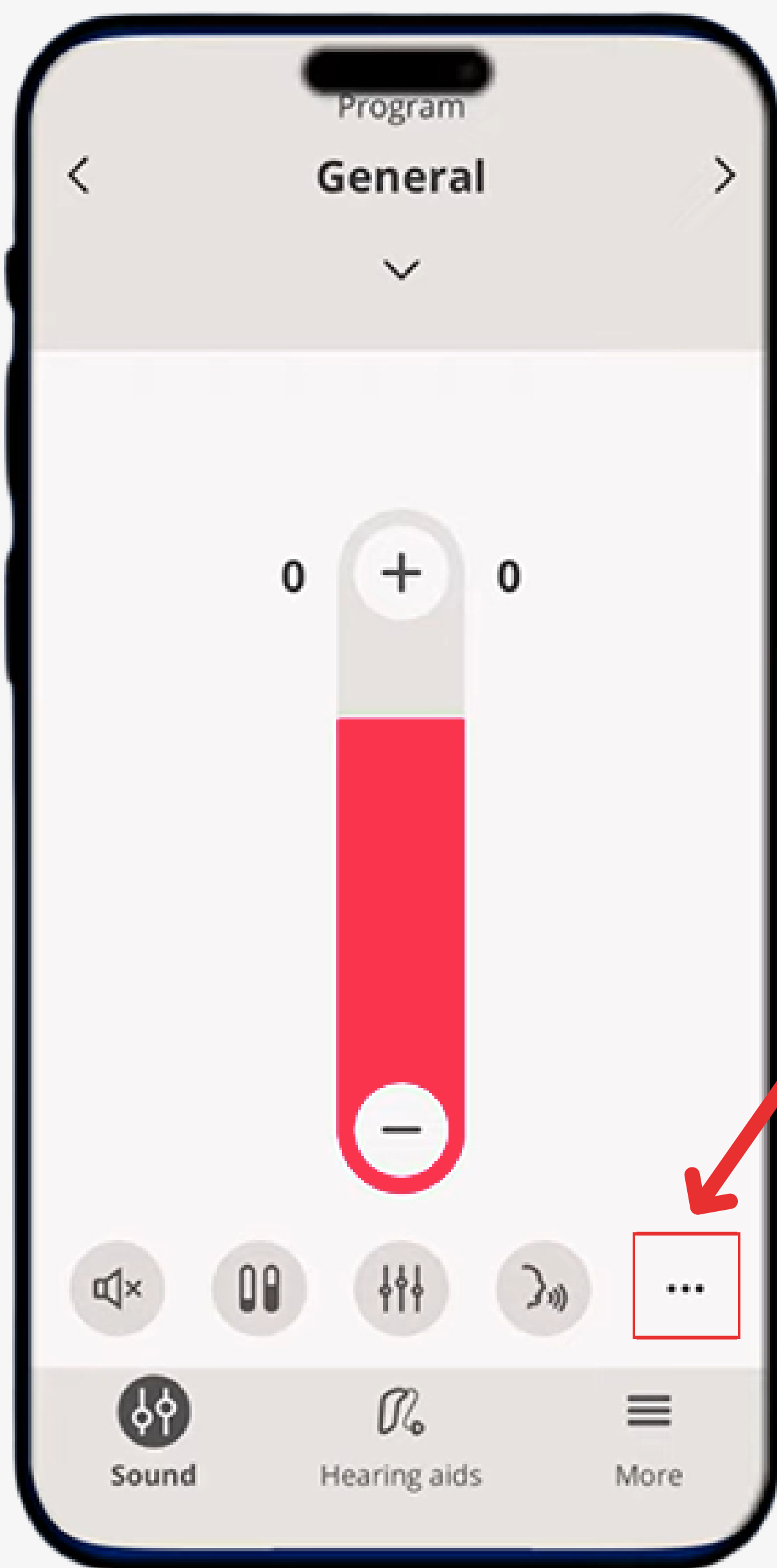


Next

< Back

Start  
Over



### ③ Tap **Auracast broadcast**.

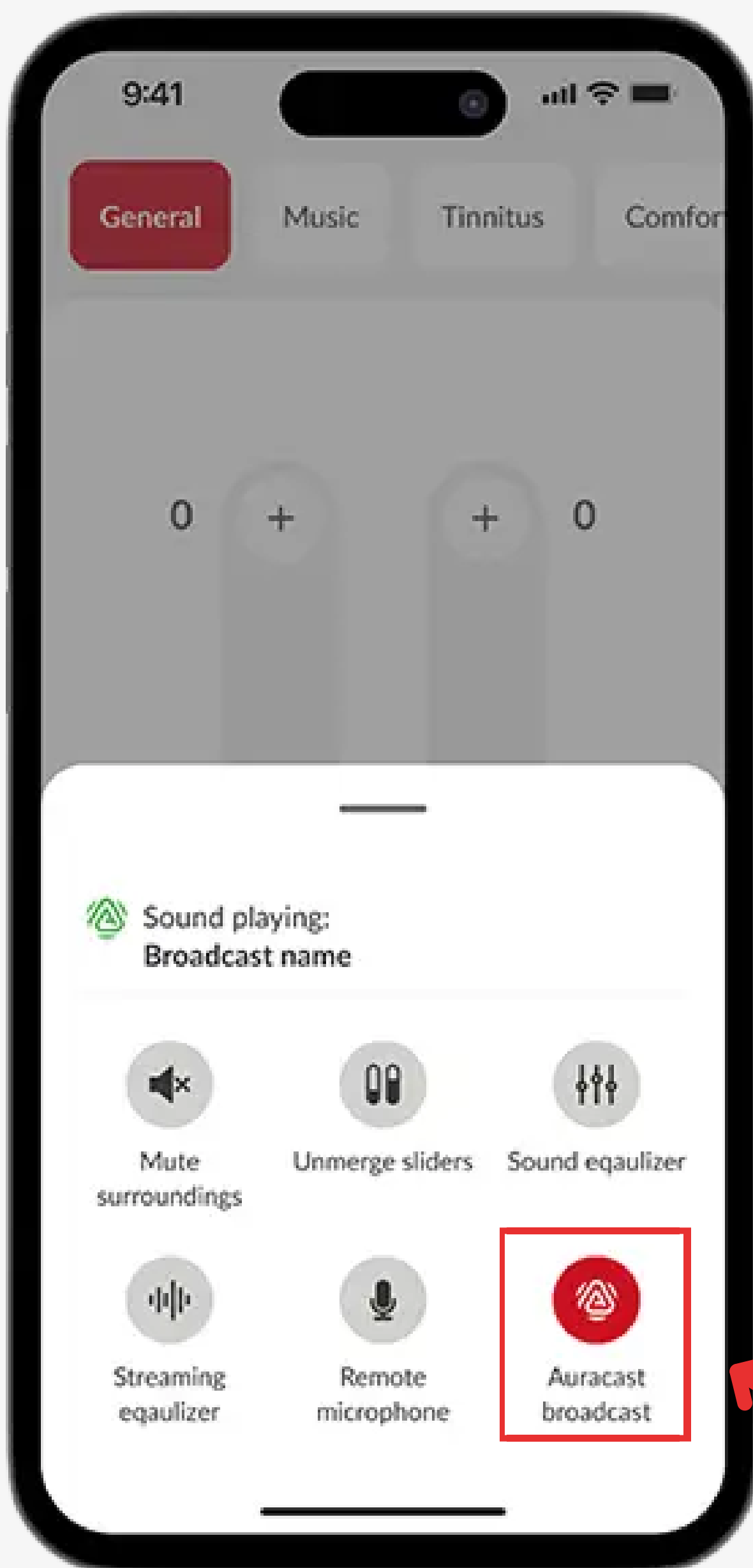


Next

< Back

Start  
Over



#### 4 Select a broadcast.

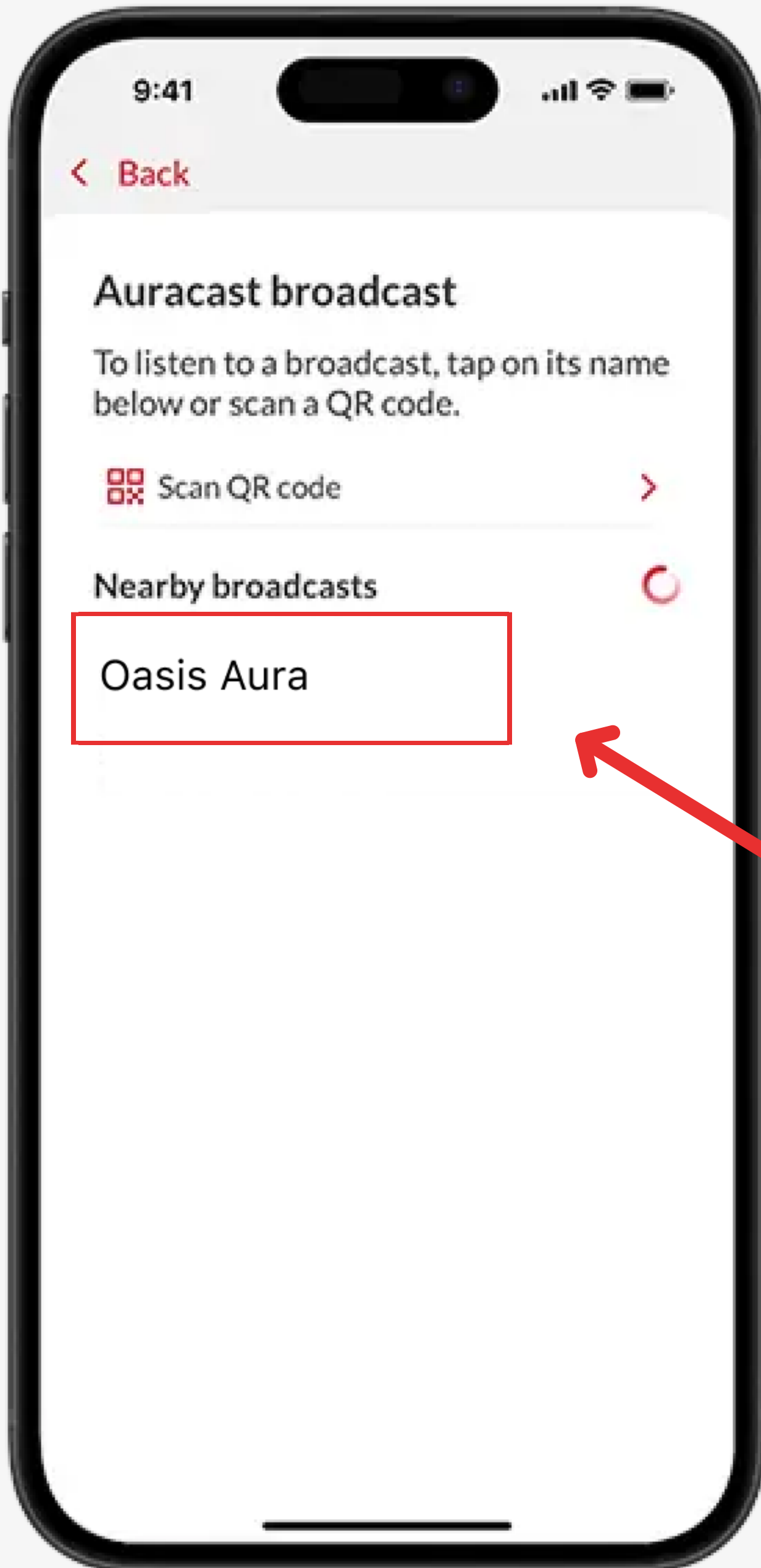


Next

< Back

Start  
Over



## 5 Adjust volume.

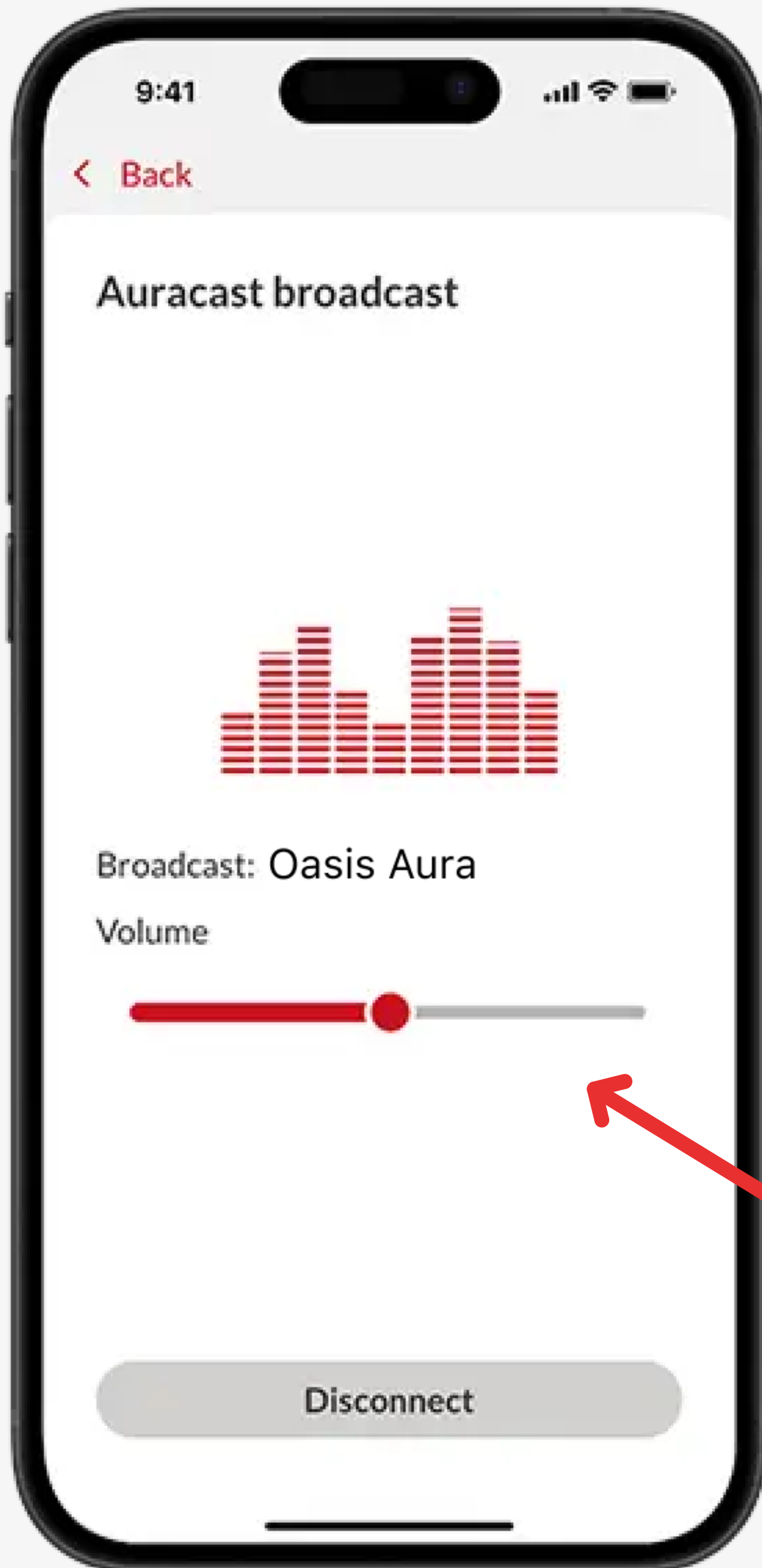

Next

< Back

Start  
Over



## Using an Auracast Receiver

Recommended receiver: [AuraClip](#)



Next

< Back

Start  
Over



## Using Auracast Headphones

You can wear Auracast headphones over your hearing aids, or use them without hearing aids.

Example: [Aria Aura](#)

Option a. Wear Over Your  
Hearing Aids



Option b. Use Without  
Hearing Aids



Next

< Back

Start  
Over



# Troubleshooting

## Common Issues

---

- No sound
- Not hearing clearly
- Hearing echo or delay

 [More FAQ & Help](#) ↗

i

*Still need help?*

[Contact support](#) ↗

Next

< Back

Start  
Over



## Rate This Guide

Your feedback helps us improve this guide for everyone.

It only takes a moment.

**Rate This Guide**