



Auracast Listening Guide for **Jabra** Hearing Aids



Learn how to listen
to Auracast broadcasts

Start

< Back

Choose your device

For **Auracast** Hearing Aids:

- **Enhance Pro 20** (*Firmware \geq v10.62*)
- **Enhance Pro 30**



Use
Jabra app



For **non-Auracast** Hearing Aids:



Use an
Auracast receiver



Use
Auracast headphones



< Back

Start
Over



Using Jabra App

- 1 Open **Jabra Enhance Pro** app on your phone.



You should see a screen like:

- “All-Around”
- Volume slider
- Sound Equalizer

Stay on this screen.

Next

< Back

Start
Over



② Wait for broadcast.

When an Auracast broadcast is available, The app may:

- Show a **streaming / connection option**
- OR automatically detect nearby audio

Wait a few seconds.

Next

< Back

Start
Over



③ Look for Streaming Option.

Check for any of these
(varies by version):

- Streaming
- Audio
- Broadcast
- Connection icon

Usually appears at Bottom area / Top
icons / pop-up.


Next

< Back

Start
Over



4 Select a broadcast.

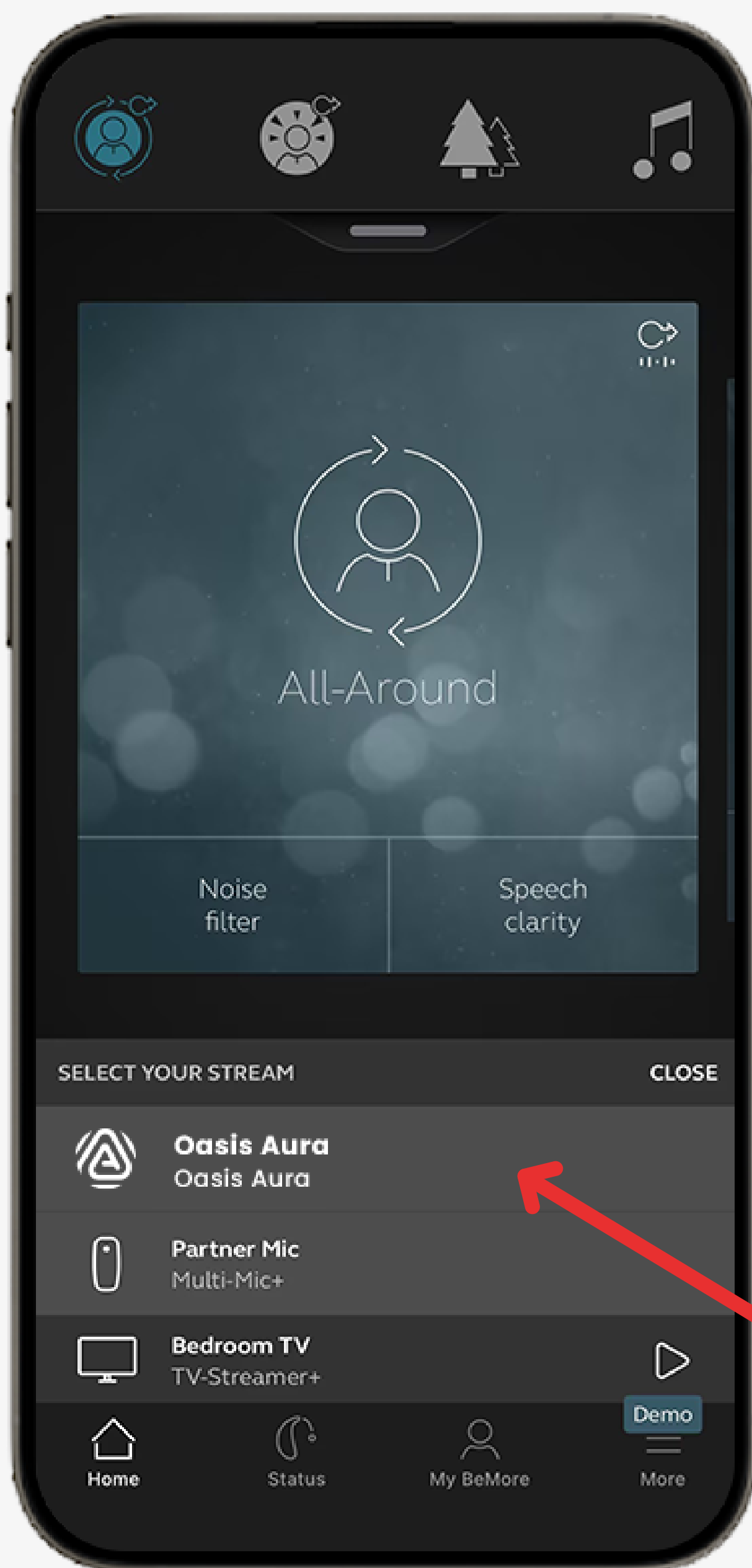


Next

< Back

Start
Over



5 Adjust volume.

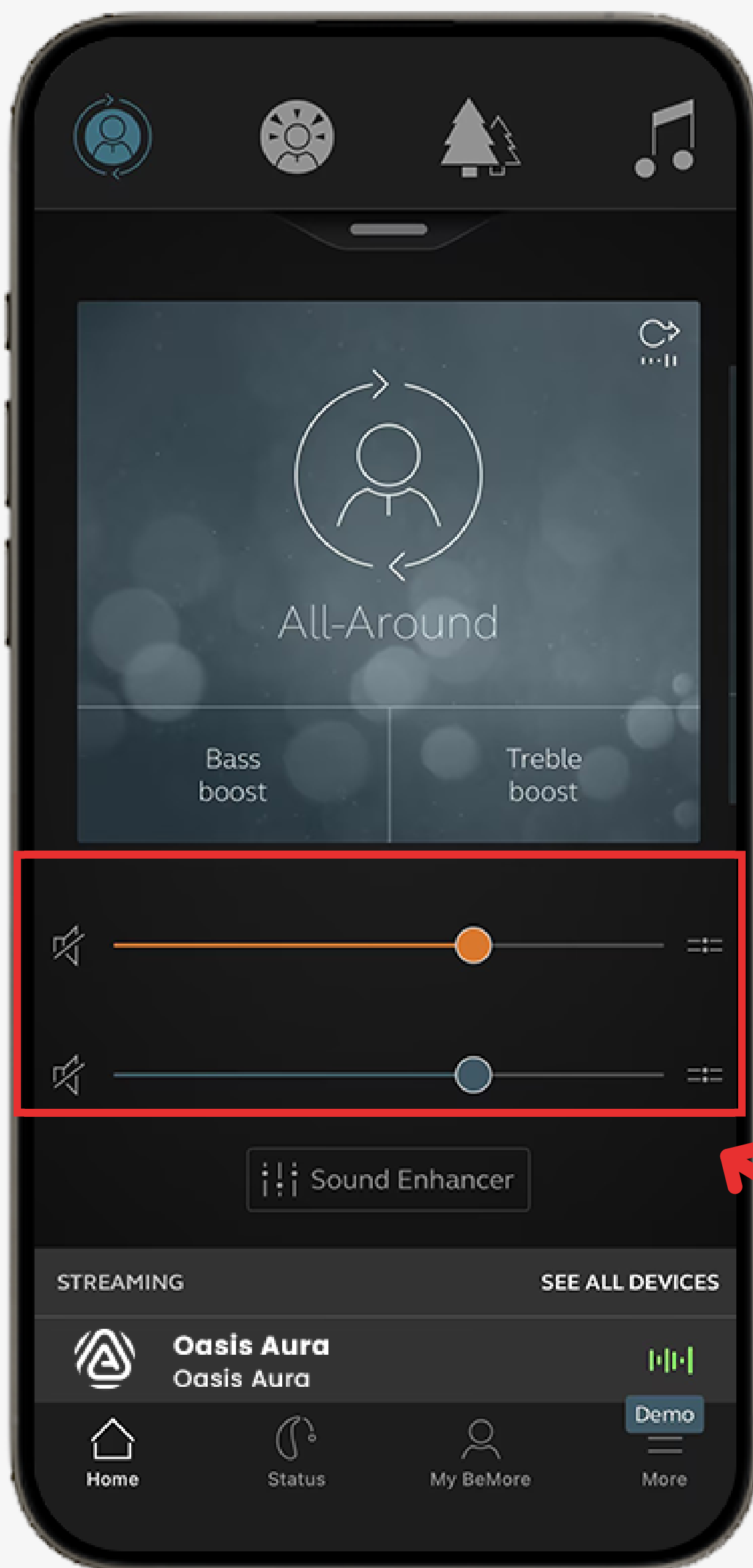

Next

< Back

Start
Over



Using an Auracast Receiver

Recommended receiver: [AuraClip](#)



Next

< Back

Start
Over



Using Auracast Headphones

You can wear Auracast headphones over your hearing aids, or use them without hearing aids.

Example: **Aria Aura**

Option a. Wear Over Your
Hearing Aids



Option b. Use Without
Hearing Aids



Next

< Back

Start
Over



Troubleshooting

Common Issues

- No sound
- Not hearing clearly
- Hearing echo or delay

 [More FAQ & Help ↗](#)



Still need help?
[Contact support](#) ↗

Next

< Back

Start
Over



Rate This Guide

Your feedback helps us improve this guide for everyone.

It only takes a moment.

Rate This Guide