



Auracast Listening Guide for **Oticon** Hearing Aids



Learn how to listen
to Auracast broadcasts

Start

< Back

Choose your device

For **Auracast** Hearing Aids:

- *Intent*
- *Zeal*



Use
Oticon app



For **non-Auracast** Hearing Aids:



Use an
Auracast receiver



Use
Auracast headphones



< Back

Start
Over



Using Oticon App

1 Open **Oticon app (latest version)** on your phone.



*Using Intent? Update to firmware **1.3.0 or newer.***



Next

< Back

Start
Over



② Tap ... (More Options) button.



Next

< Back

Start
Over



3 Tap **Auracast broadcast**.




Next

< Back

Start
Over



4 Select a broadcast.

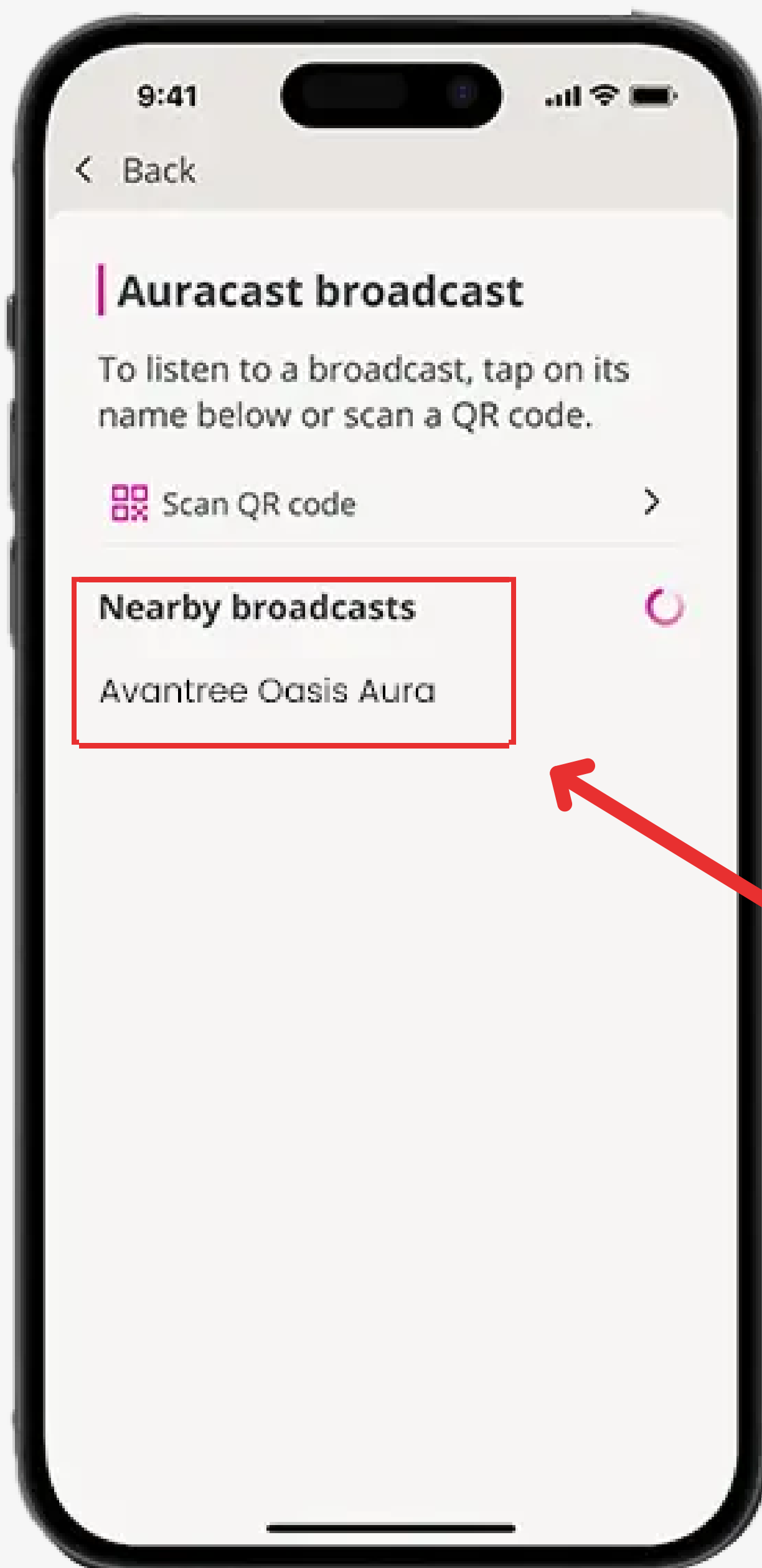


Next

< Back

Start
Over



5 Adjust volume.

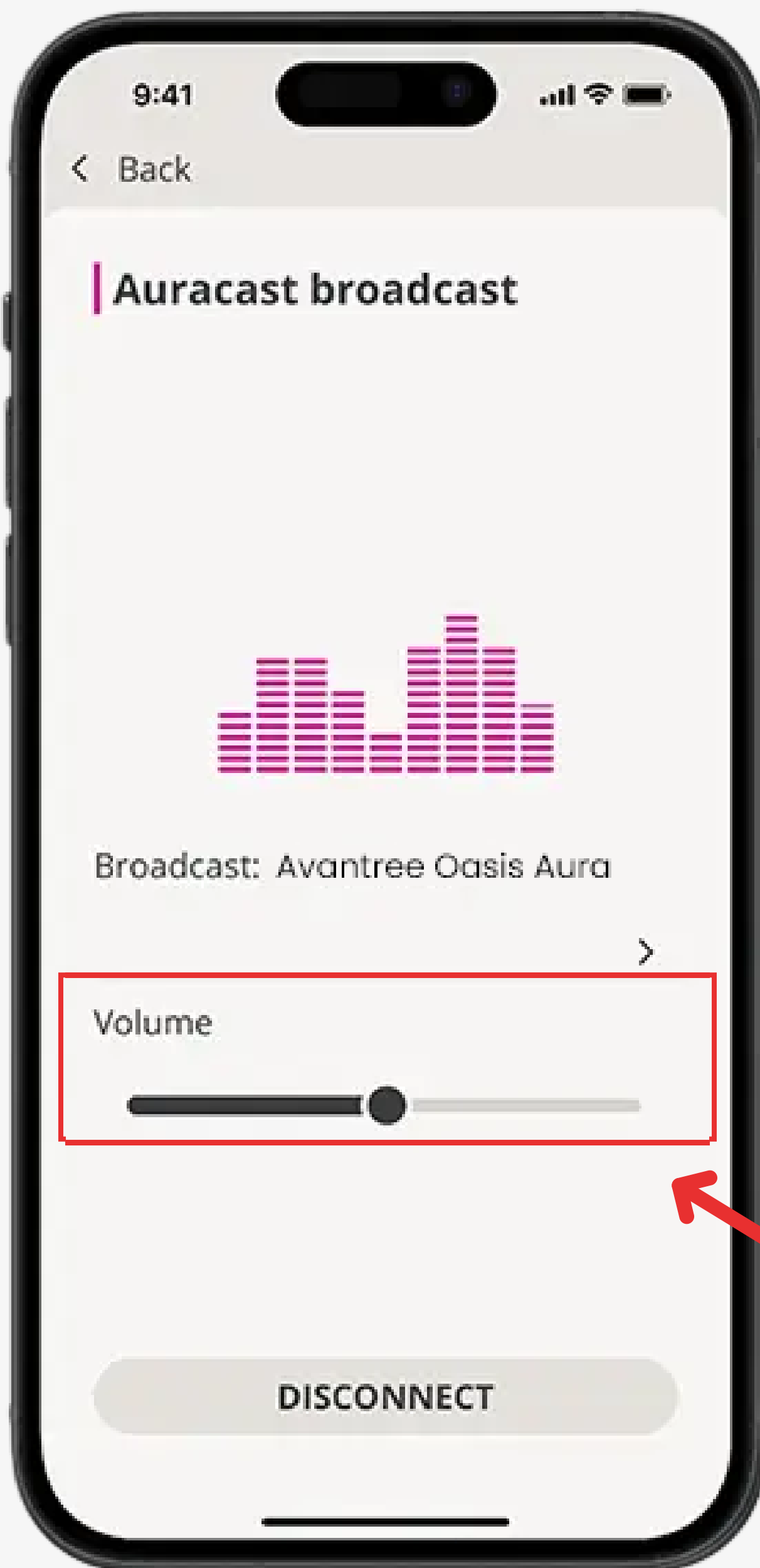

Next

< Back

Start
Over



Using an Auracast Receiver

Recommended receiver: [AuraClip](#)



Next

< Back

Start
Over



Using Auracast Headphones

You can wear Auracast headphones over your hearing aids, or use them without hearing aids.

Example: [Aria Aura](#)

Option a. Wear over your hearing aids



Option b. Use without hearing aids



Next

< Back

Start
Over



Troubleshooting

Common Issues

- No sound
- Not hearing clearly
- Hearing echo or delay

 [More FAQ & Help ↗](#)



Still need help?
[Contact support ↗](#)

Next

< Back

Start
Over



Rate This Guide

Your feedback helps us improve this guide for everyone.

It only takes a moment.

Rate This Guide