



# Auracast Listening Guide for **Resound** Hearing Aids

---



Learn how to listen  
to Auracast broadcasts

Start

< Back

# Choose your device

For **Auracast** Hearing Aids:

- *Enzo IA*
- *Savi*
- *Nexia*
- *Vivia*



Use  
**ReSound app**



For **non-Auracast** Hearing Aids:



Use an  
**Auracast receiver**



Use  
**Auracast headphones**



< Back

Start  
Over



# Using ReSound App

- 1 Open **ReSound Smart 3D** app (*latest version*) on your phone.




Next

< Back

Start  
Over



② Tap **Search** or **SEE ALL DEVICES**.

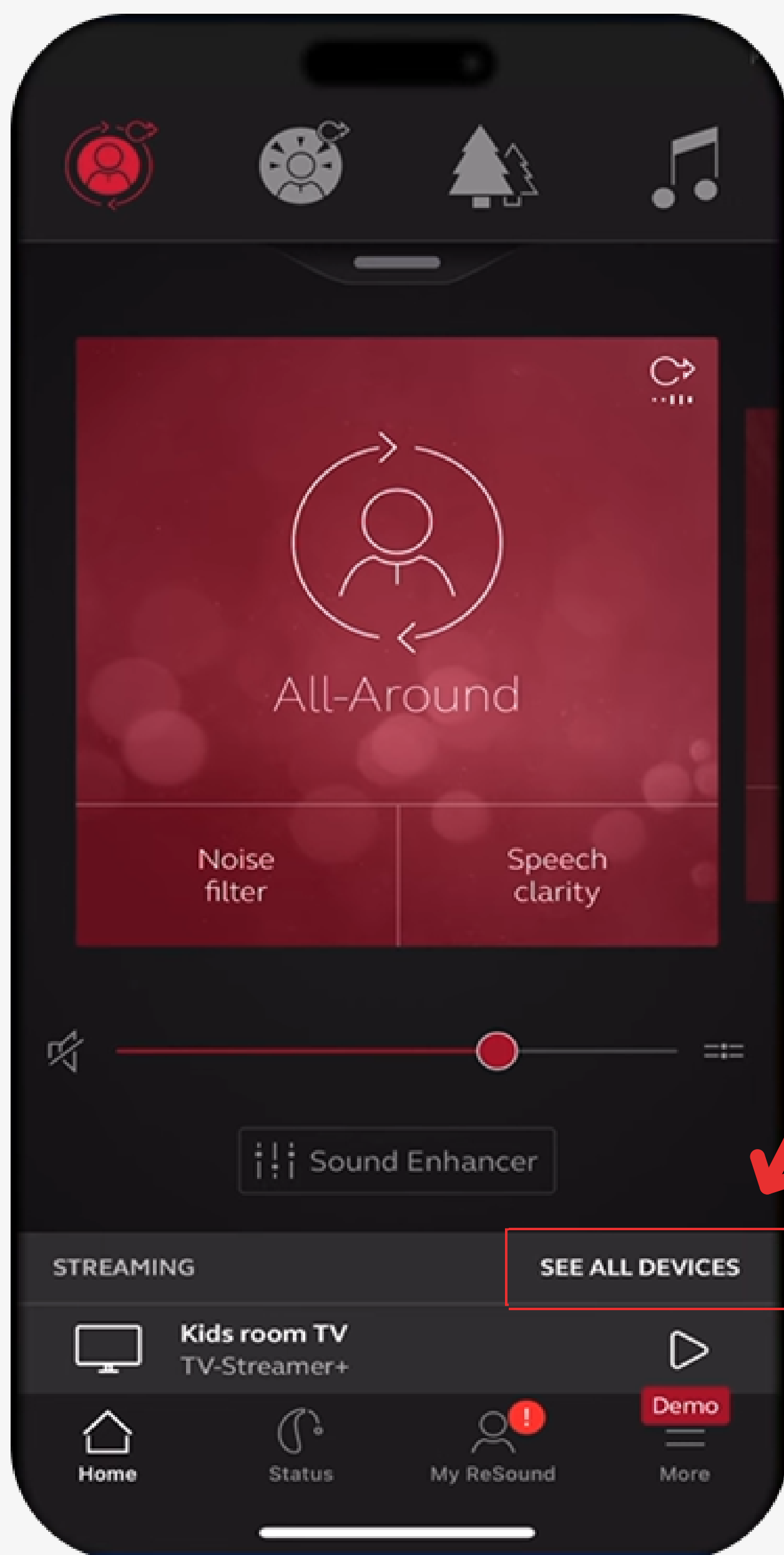


Next

< Back

Start  
Over



3 Select a broadcast.

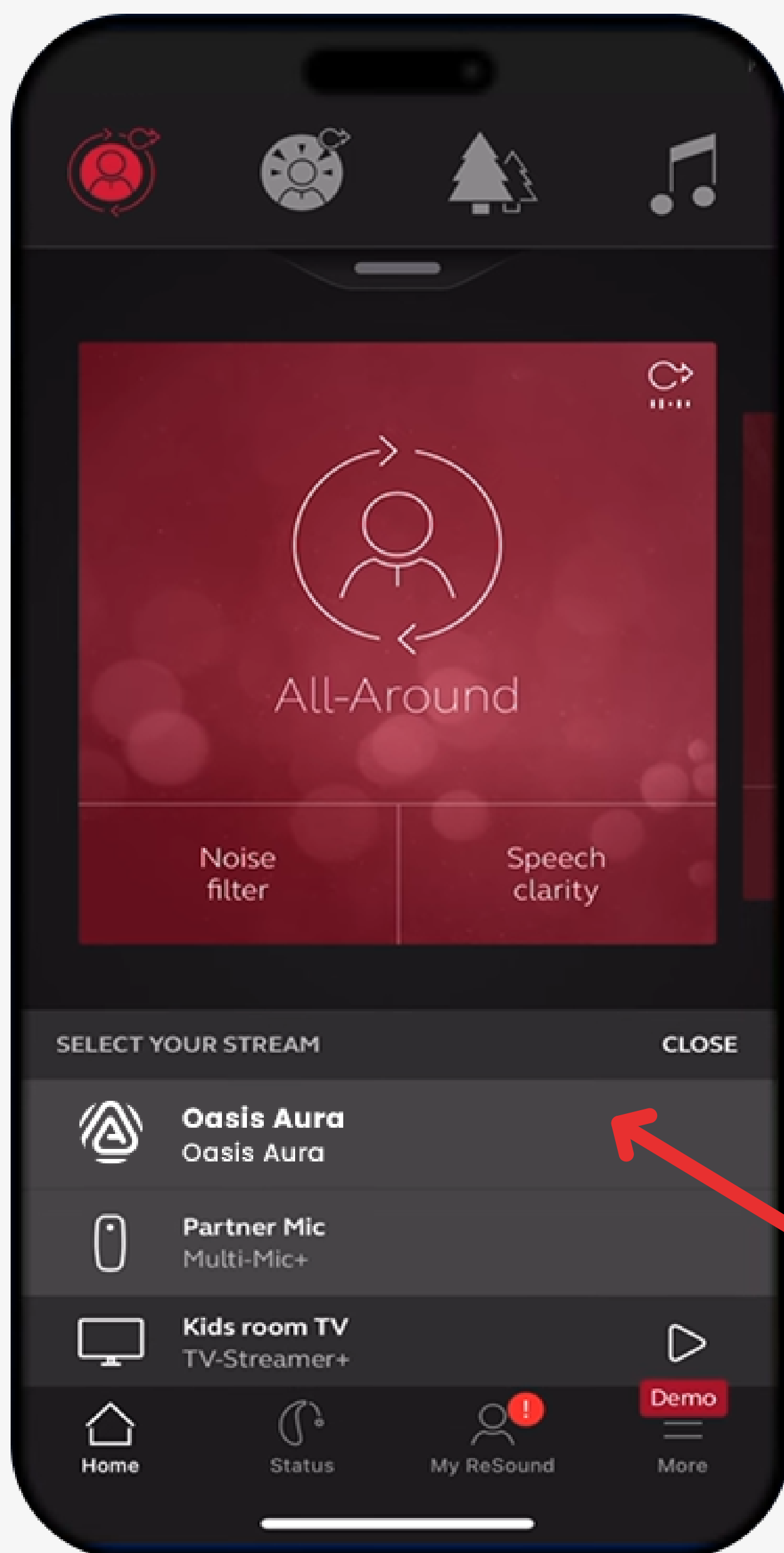


Next

< Back

Start  
Over



## 4 Adjust volume.

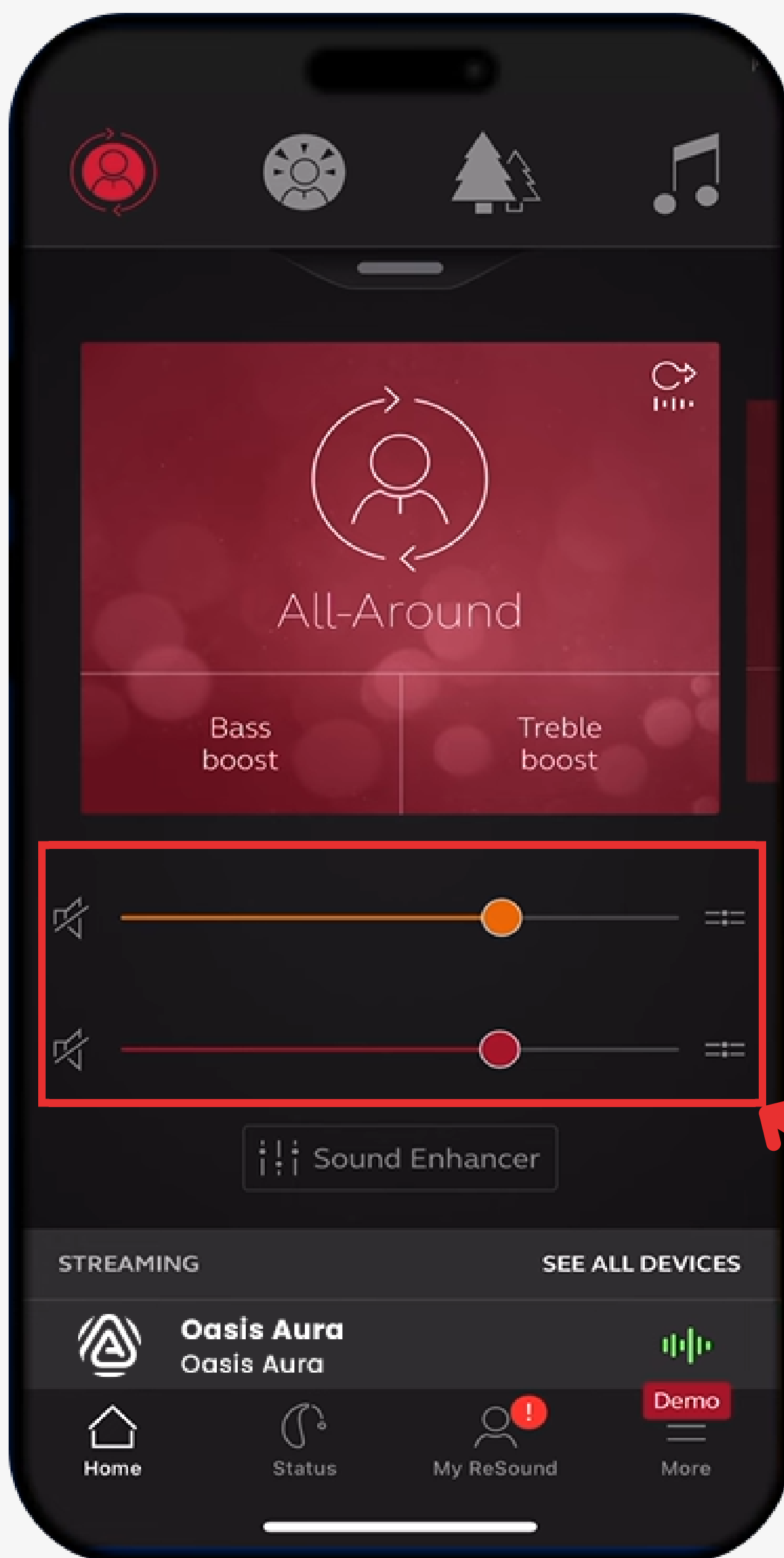

Next

< Back

Start  
Over



## Using an Auracast Receiver

Recommended receiver: [AuraClip](#)



Next

< Back

Start  
Over



## Using Auracast Headphones

You can wear Auracast headphones over your hearing aids, or use them without hearing aids.

Example: [Aria Aura](#)

Option a. Wear Over Your  
Hearing Aids



Option b. Use Without  
Hearing Aids



Next

< Back

Start  
Over



# Troubleshooting

## Common Issues

---

- No sound
- Not hearing clearly
- Hearing echo or delay

 [More FAQ & Help](#) ↗



*Still need help?*

[Contact support](#) ↗

Next

< Back

Start  
Over



# Rate This Guide

Your feedback helps us improve this guide for everyone.

It only takes a moment.

**Rate This Guide**