



Auracast Listening Guide for **Starkey** Hearing Aids



Learn how to listen
to Auracast broadcasts

Start

< Back

Choose your device

For **Auracast** Hearing Aids:

- *Edge AI*
- *Omega AI*



Use
Starkey app



For **non-Auracast** Hearing Aids:



Use an
Auracast receiver



Use
Auracast headphones



< Back

Start
Over



Using Starkey App

- 1 Open **My Starkey** app
(*Version 6.0.0+*) on your phone.




Next

< Back

Start
Over



2 Tap **Stream options**.

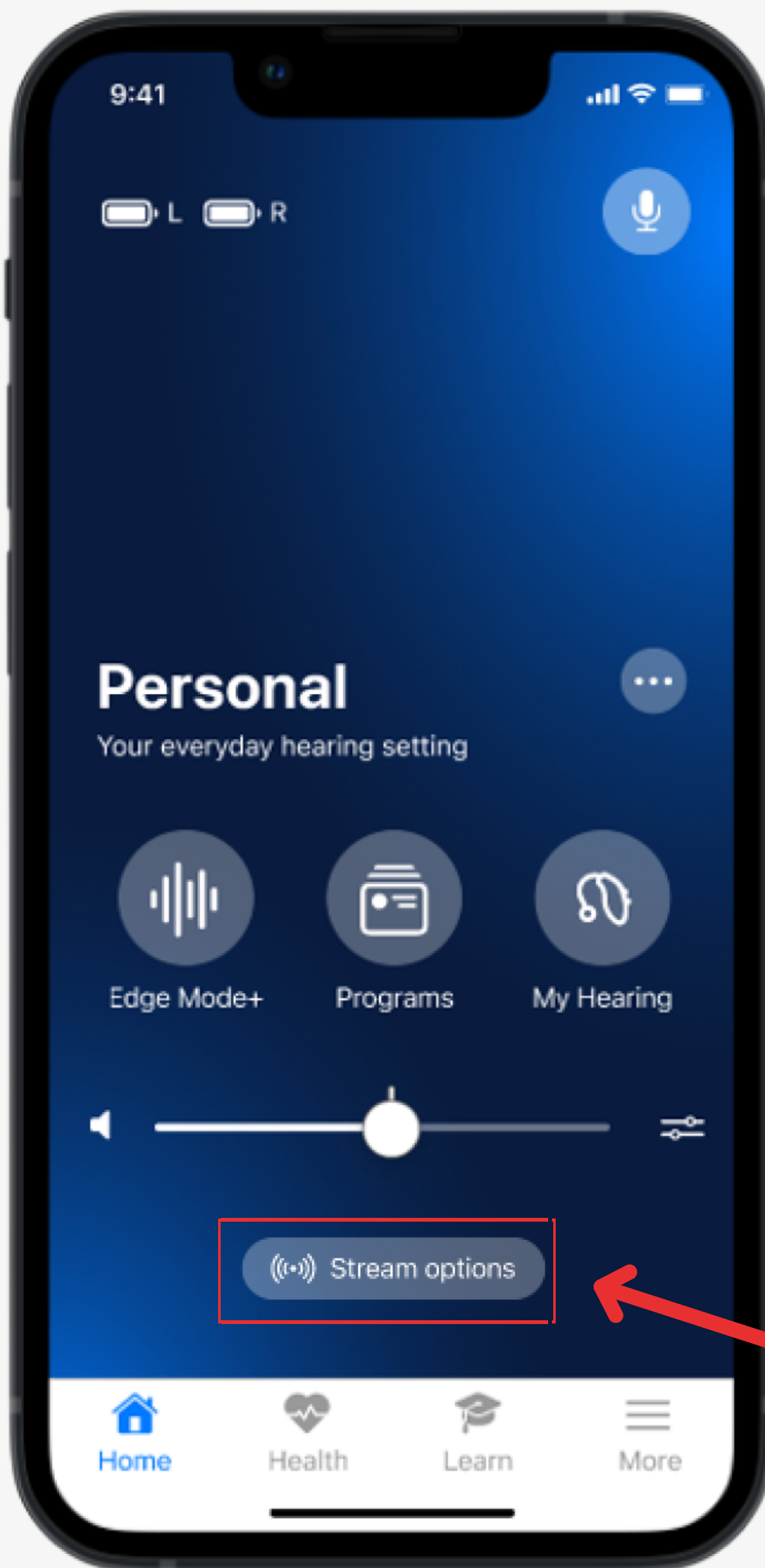


Next

< Back

Start
Over



③ Tap **Auracast broadcasts**.

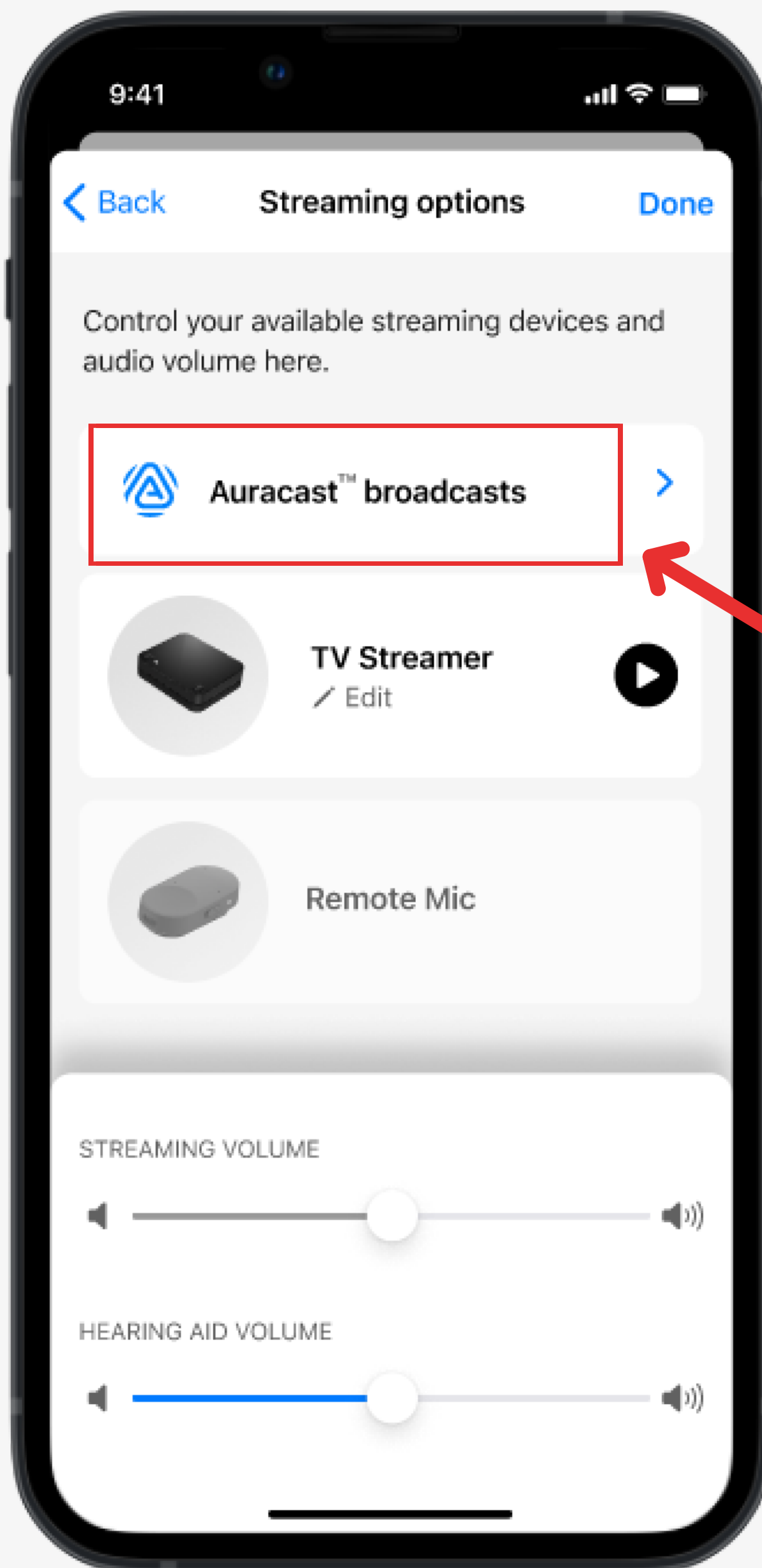


Next

< Back

Start
Over



4 Select a broadcast.

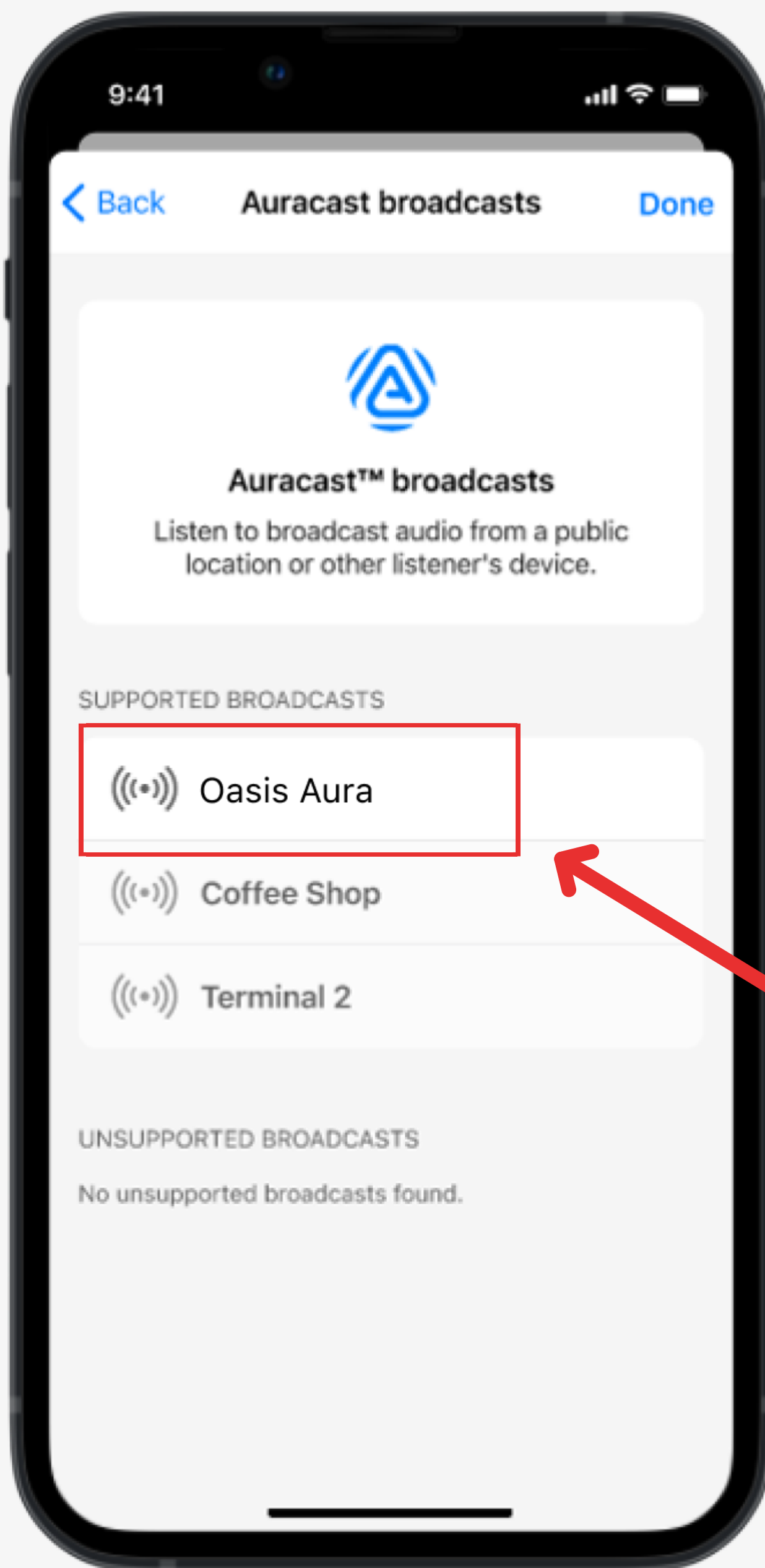


Next

< Back

Start
Over



5 Adjust volume.

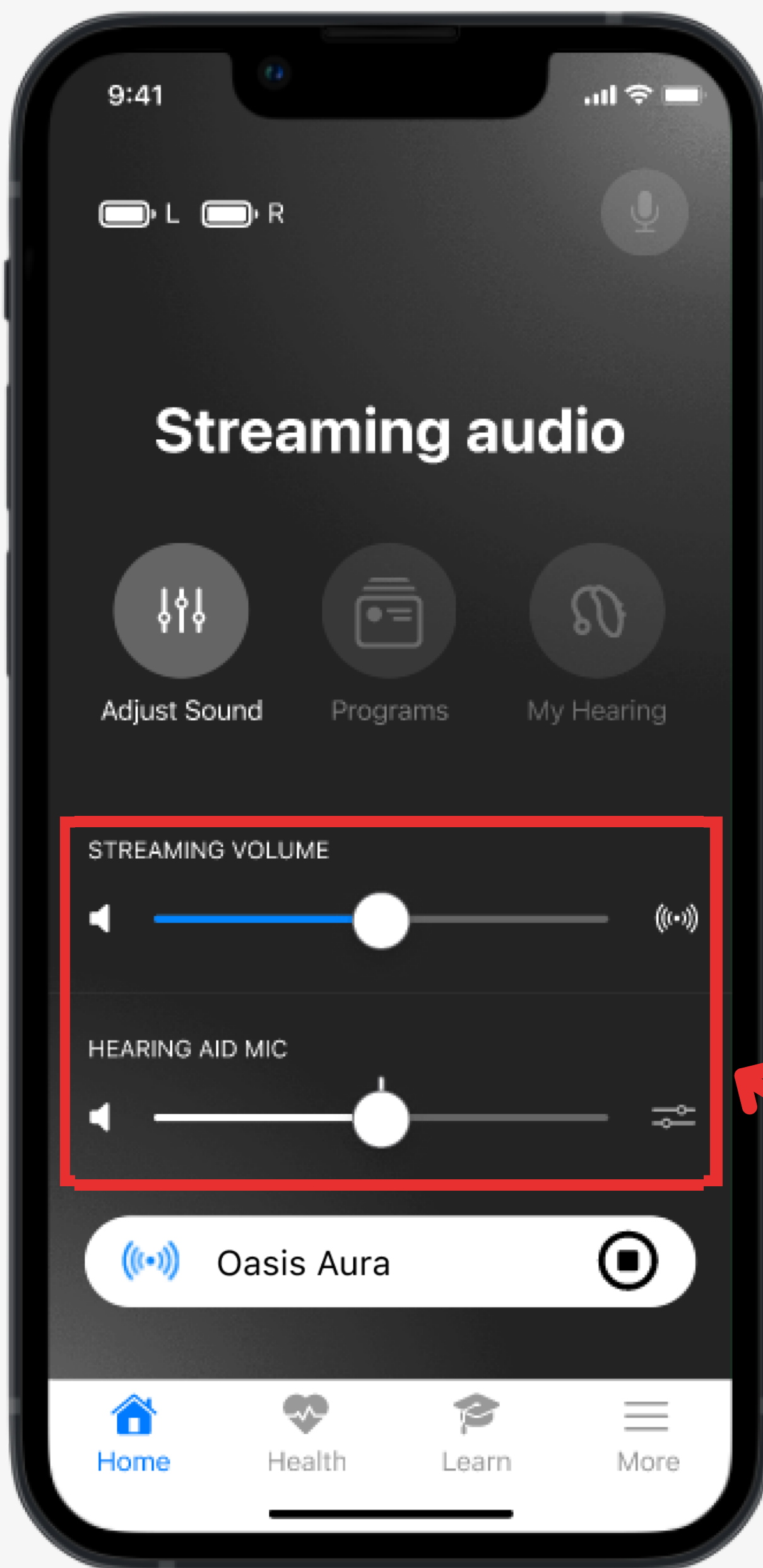

Next

< Back

Start
Over



Using an Auracast Receiver

Recommended receiver: [AuraClip](#)



Next

< Back

Start
Over



Using Auracast Headphones

You can wear Auracast headphones over your hearing aids, or use them without hearing aids.

Example: [Aria Aura](#)

Option a. Wear Over Your
Hearing Aids



Option b. Use Without
Hearing Aids



Next

< Back

Start
Over



Troubleshooting

Common Issues

- No sound
- Not hearing clearly
- Hearing echo or delay

 [More FAQ & Help](#) ↗



Still need help?

[Contact support](#) ↗

Next

< Back

Start
Over



Rate This Guide

Your feedback helps us improve this guide for everyone.

It only takes a moment.

Rate This Guide