



Auracast Listening Guide for **Widex** Hearing Aids



Learn how to listen
to Auracast broadcasts

Start

< Back

Choose your device

For **Auracast** Hearing Aids:

Allure
(firmware \geq 5.1.20)



Use
Auracast Assistant



For **non-Auracast** Hearing Aids:



Use an
Auracast receiver



Use
Auracast headphones



< Back

Start
Over




Join with Auracast Assistant

- **Available on newer Android phones.**
[Check supported phones](#) ↗

Example: Samsung Galaxy S24

- ➊ Connect hearing aids with phone.
- ➋ Open phone Bluetooth setting.
- ➌ Tap **Listen to Auracast broadcast**.
- ➍ Select a broadcast.

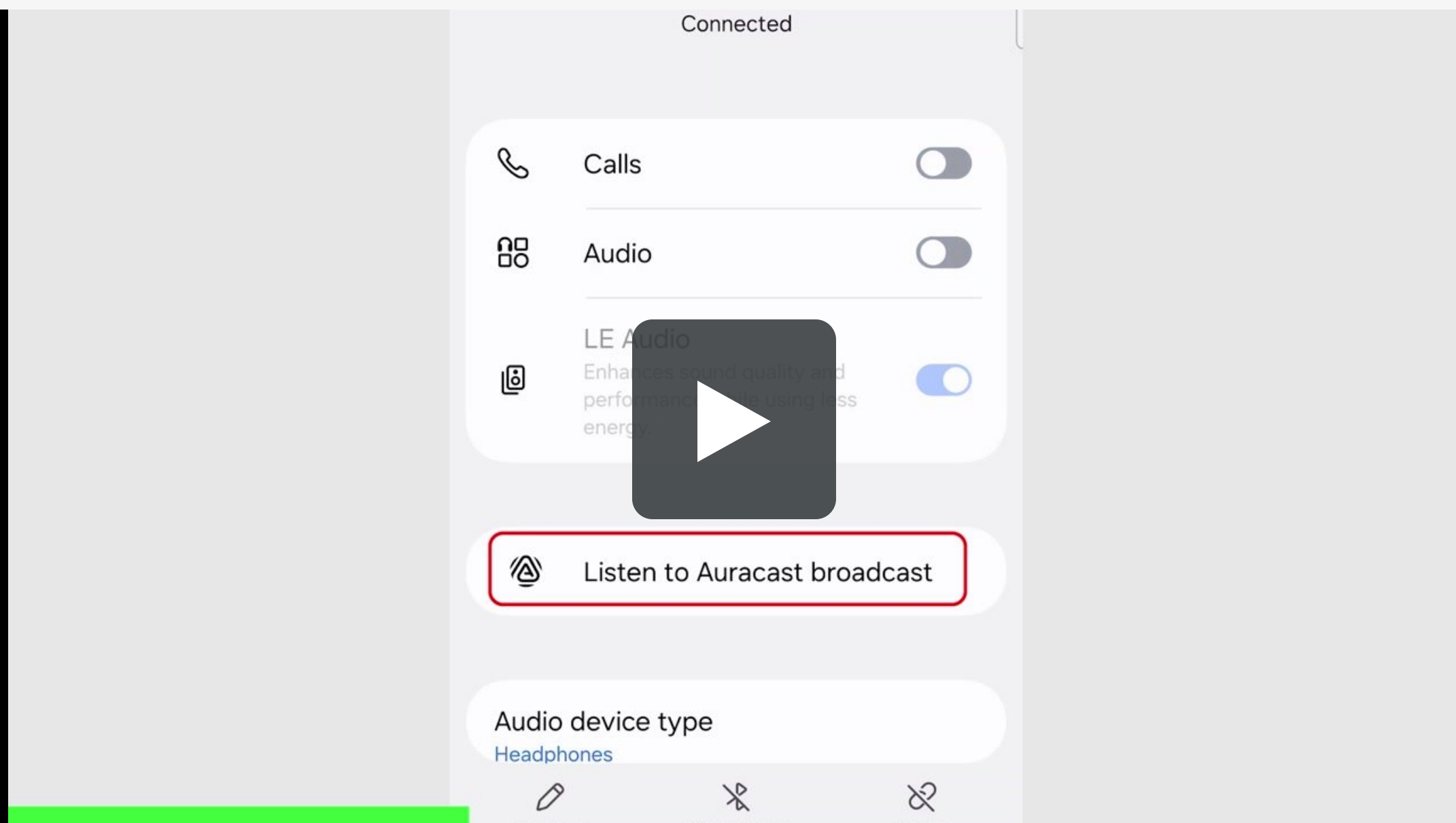

Next

< Back

Start
Over



Using an Auracast Receiver

Recommended receiver: [AuraClip](#)



Next

< Back

Start
Over



Using Auracast Headphones

You can wear Auracast headphones over your hearing aids, or use them without hearing aids.

Example: [Aria Aura](#)

Option a. Wear Over Your
Hearing Aids



Option b. Use Without
Hearing Aids



Next

< Back

Start
Over



Troubleshooting

Common Issues

- No sound
- Not hearing clearly
- Hearing echo or delay

 [More FAQ & Help](#) ↗



Still need help?

[Contact support](#) ↗

Next

< Back

Start
Over



Rate This Guide

Your feedback helps us improve this guide for everyone.

It only takes a moment.

[Rate This Guide](#)