



Auracast Listening Guide for **Danalogic** Hearing Aids



Learn how to listen
to Auracast broadcasts

Start

< Back

Choose your device

For **Auracast** Hearing Aids:

Extend



Use
Danalogi app



For **non-Auracast** Hearing Aids:



Use an
Auracast receiver



Use
Auracast headphones



< Back

Cover

Using Danalogi App

- 1 Open **Danalogi BeMore** app on your phone.




Next

< Back

Cover

② Tap **SEE ALL DEVICES**.

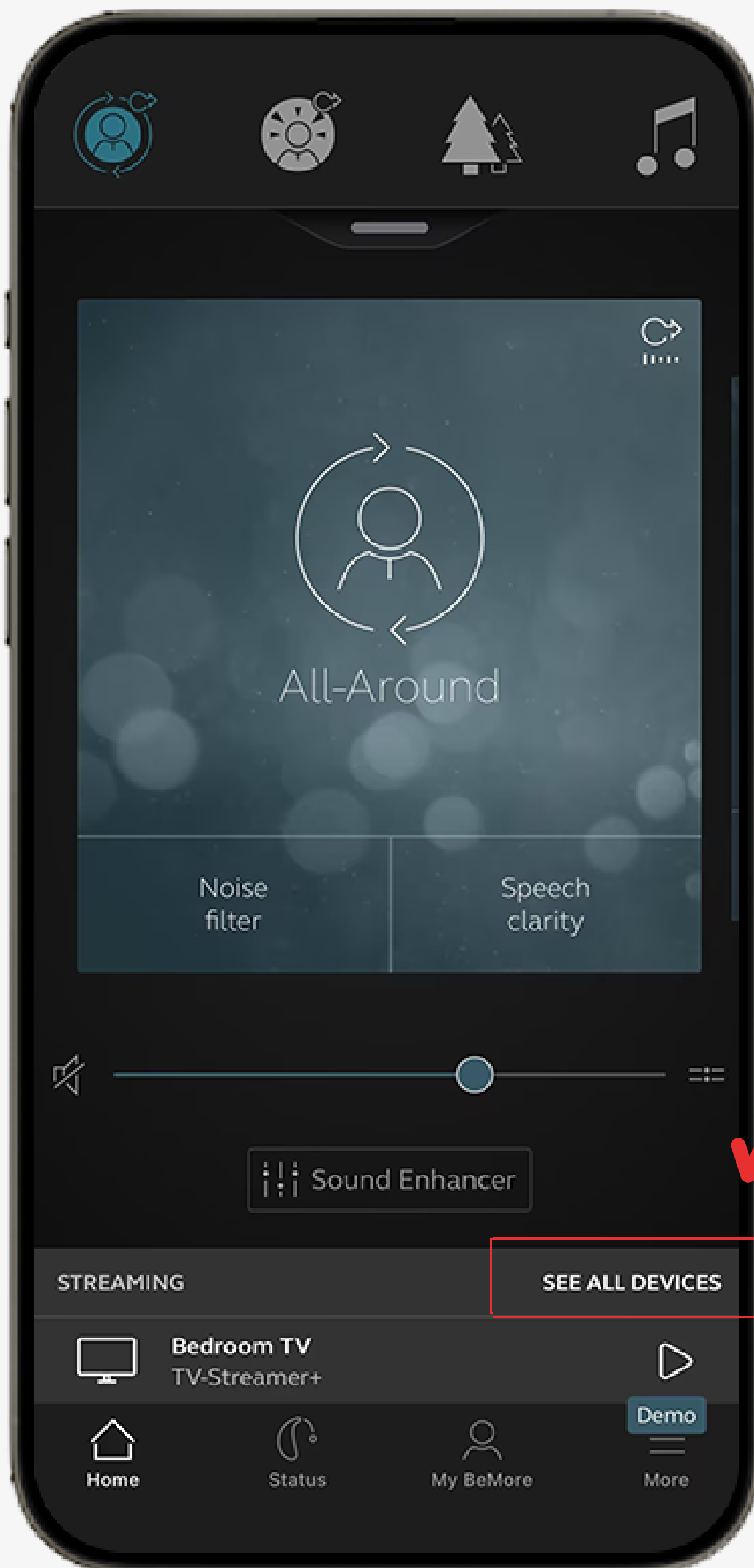


Next

< Back

Cover

③ Select a broadcast.

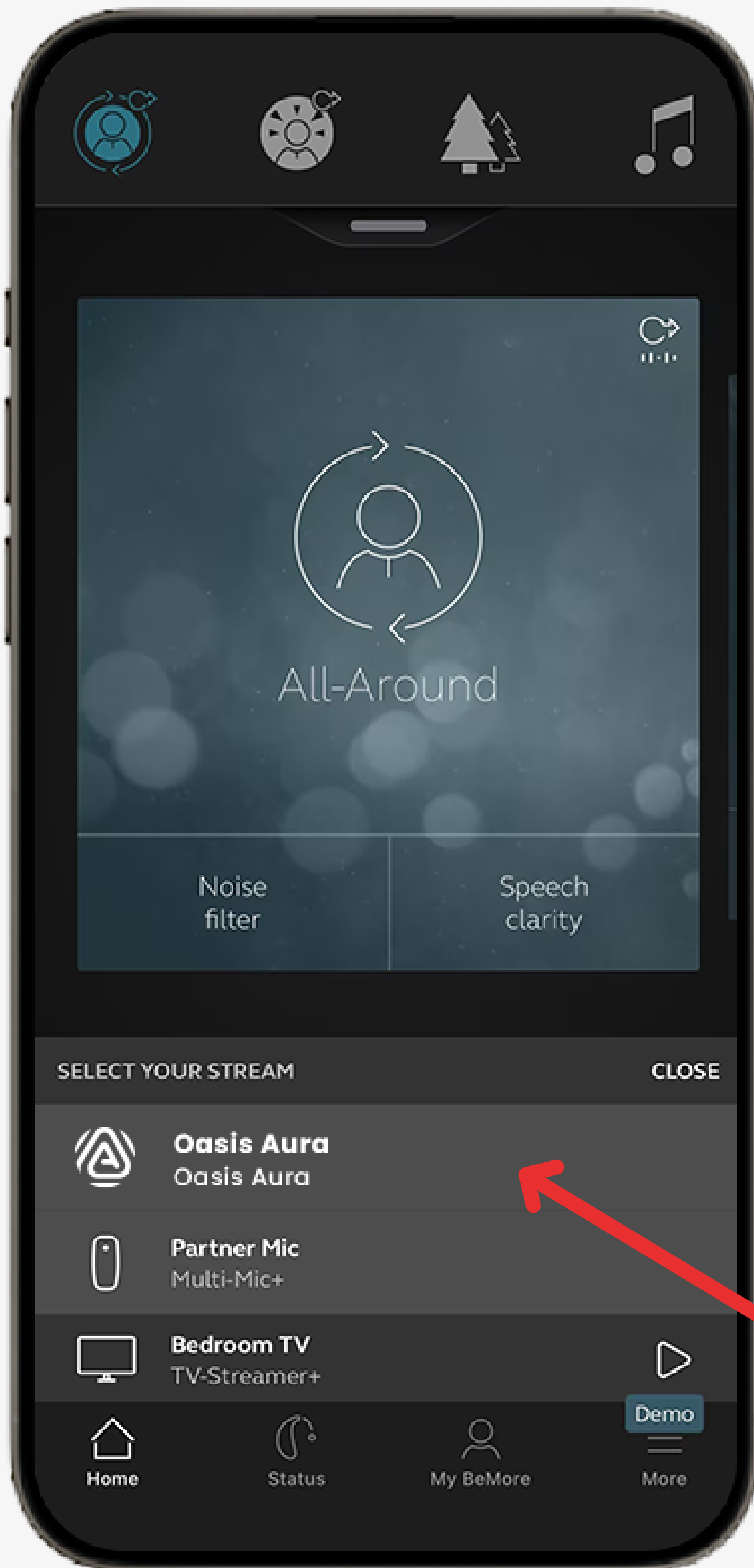


Next

< Back

Cover

4 Adjust volume.

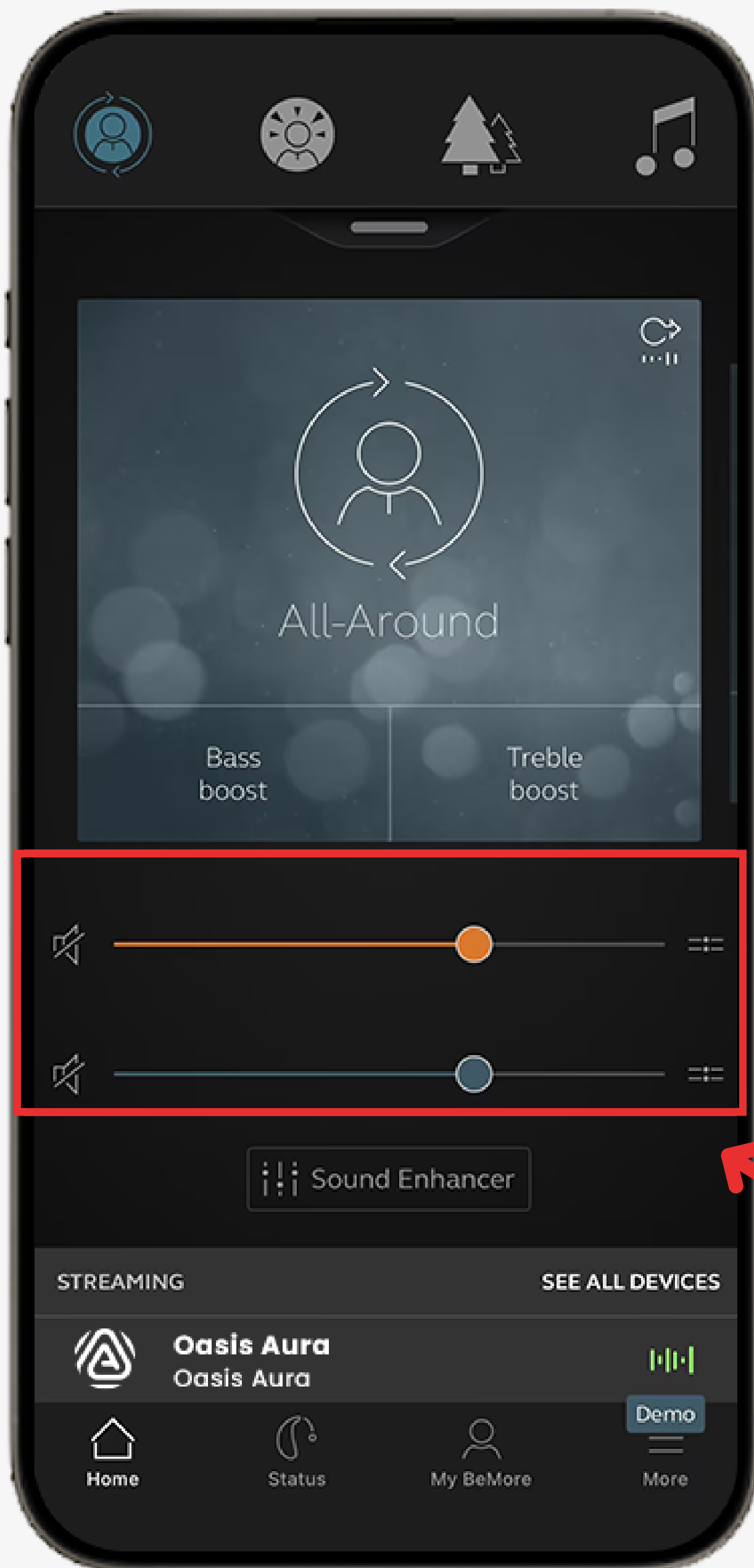

Next

< Back

Cover

Using an Auracast Receiver

Recommended receiver: [AuraClip](#)



Next

< Back

Cover

Using Auracast Headphones

You can wear Auracast headphones over your hearing aids, or use them without hearing aids.

Example: [Aria Aura](#)

Option a. Wear Over Your Hearing Aids



Option b. Use Without Hearing Aids



Next

< Back

Cover

Troubleshooting

Common Issues

- No sound
- Not hearing clearly
- Hearing echo or delay

 [More FAQ & Help ↗](#)



Still need help?

[Contact support ↗](#)

Next

< Back

Cover

Rate This Guide

Your feedback helps us improve this guide for everyone.

It only takes a moment.

Rate This Guide